

Simple, safety & quality for everyone

CAPJECT™



Simple

- Three steps
- Ergonomically shaped for comfortability
- Single-hand usage
- Reduced training time

Safe

- One-time usage, zero infection risk
- Automatic retraction of lancet
- Visibility for precise placement
- ISO 11137 standard sterilization

Safety Lancets Instructions For Use



Step 1

Pull

After cleaning puncture site, twist off protective tab for needle or pull off for blade and place in disposal.



Step 2

Place

Position lancet properly on puncture site as illustrated.



Step 3

Press

Firmly press the white button with thumb to activate lancet. Dispose of lancet in appropriate sharps container.



Step 4

Gently massage from the hand to near the puncture site to obtain the required blood volume.

Refer to the product labels and package insert for complete warnings, precautions, potential complications, and instructions for use.

Product code	200106	Depth	1.25mm
Type	Single usage, push-button type	Lancet Color	Pink
Gauge	30G	Packaging	200 Lancets per box / 10 boxes per case –2,000 Lancets
Material	Holder: High-density polyethylene (HD-PE); Lancet: Low-density polyethylene (LD-PE)+ SUS304; Spring: Polyoxymethylene (POM)		

Terumo Vietnam Medical Equipment Co., Ltd.

14th and 16th Floor, Geleximco Building,
36 Hoang Cau Street, O Cho Dua Ward,
Dong Da District, Hanoi Vietnam
+84-24-3936-1643

Terumo Singapore Pte. Ltd.

300 Beach Road #33-06 The Concourse Singapore 199555
+65-6-291-3603

PT. Terumo Indonesia

Wisma KEIAI 5th Floor J.L. Jend.
Sudirman KAV. 3 Jakarta 10220 Indonesia
+62-21-572-4071

Terumo (Thailand) Co., Ltd.

No.88 The PARQ Building, Unit 8W9-16, 8th Floor,
Ratchadaphisek Road, Klongtoey Subdistrict, Klongtoey District,
Bangkok 10110 Thailand
+66 2111 2133

Terumo Marketing Philippines Inc.

3203 West Tower, Pse Centre Exchange Road, Ortigas Center
Pasig City, Philippines
+63-2-8632-1674

Terumo Singapore Pte. Ltd. (Yangon Branch)

Room no. 1220, 12th Floor, Myanmar Centre Business Suites,
Myanmar Centre Tower 2, 192 Kaba Aye Pagoda Road,
Bahan Township, Yangon, Myanmar
+95-1-934-5065 (Ext.1228)

Not all specifications which are described in this catalog are registered and approved for sale in all countries or regions. Indications of use may also vary by country and region. Please contact your country representative for product availability and information.